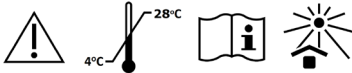
i-PRO[®] LC

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INSTRUCTION FOR USE EN

DESCRIPTION

i-PRO LC is a light curing material for temporary use based on resin and inorganic filler. i-PRO LC has easily packable consistency. This material is ideal for temporary fillings of short and extended periods.

INSTRUCTION FOR USE

- NOTE!** Do not use if primary package is broken.
- Using suitable instrument take out a little of the material from the syringe and immediately close syringe cap.
 - Form a ball and place direct into cavity and adapt as necessary.
 - Do not apply layers more than 2 mm thickness.
 - Light curing for 20-40 seconds depending on layer thickness.

NOTE! After the desired amount of material extruded, the syringe cap must be closed immediately, so that the material is not unlighted.

WARNINGS AND PRECAUTIONS!

NOTE! Product contains urethane acrylate. Uncured product may cause sensitization by skin contact, may cause allergic reactions. Do not use i-PRO LC if patient is allergic to product or to any of the ingredients.

Hazard and precautionary statements:
Product causes skin, eye, and respiratory irritation. IF ON SKIN: Wash with plenty of soap and water. If skin irritation occurs: Get medical advice/attention. Take off contaminated clothing and wash before reuse. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention.

Prevention:
Wash hands thoroughly after handling. Use only in a well ventilated area. Wear protective gloves/protective clothing/eye protection/face protection.

Interaction with other substances:
i-PRO LC should not be used with products containing eugenol.

STORAGE

Keep container tightly closed in dry well-ventilated place at 4-28°C. Protect from direct sunlight and heat sources. The batch number should be quoted in all correspondence. See packaging for batch and expiry date. Do not use after the expiry date. Dispose of contents/container to as required by national regulatory requirements. Keep out of the reach of children!

MANUFACTURERS RESPONSIBILITY

Our products have been developed for professional use in dentistry. As the application of our products is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our products in accordance with the applied standards.

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