
- EN** Temporary filling material for temporary filling and sealing of cavities and temporary fixing of post crowns
- ES** Material de relleno temporal para el llenado y el sellado temporal de cavidades y la fijación temporal de coronas
- LT** Laikinas dantų ertmių užpildas laikinam dantų ertmių užpildymui ir laikinam vainikėlių cementavimui
- PL** Tymczasowy cement wypełniający do wypełnień tymczasowych i tymczasowego mocowania koron
- RO** Material pentru plombarea temporară a cavităților și cimentarea temporară a coroanelor
- RU** Материал для временного заполнения полостей для и временного цементирования коронок
- TR** Geçici dolgu maddesi kaviterlerin geçici olarak doldurulmasında ve kapatılmasında ve post kronların geçici olarak sabitlenmesinde kullanılan
- UA** Матеріал для тимчасового заповнення порожнин та тимчасового цементування коронок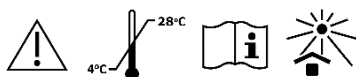

INSTRUCTION FOR USE EN

DESCRIPTION

i-PRO^N is a self-curing, eugenol-free filling for temporary use based on zinc oxide and synthetics resins. i-PRO^N has easily packable consistency, high strength and hardness.

INSTRUCTION FOR USE

NOTE! Do not use if primary package is broken.

1. Using suitable instrument take out a little of the material, form a ball with your fingers and place direct into cavity. Adapt as necessary.
2. Material hardens by absorption of moisture from mouth.
3. For provisional fixing of post crowns spread a layer on inside of crowns and seal in normal way.
4. It is recommended that masticate pressure is not applied for about 1 hour.

WARNINGS AND PRECAUTIONS!

NOTE! Do not use i-PRO^N if patient is allergic to product or to any of the ingredients.

Prevention:

Wash hands thoroughly after handling. Wear protective gloves/protective clothing/eye protection/face protection.

STORAGE

Keep container tightly closed in dry well-ventilated place at 4-28°C. Protect from direct sunlight and heat sources. Do not freeze the product. The batch number should be quoted in all correspondence. See packaging for batch and expiry date. Do not use after the expiry date. Dispose of contents/container to as required by national regulatory requirements. Keep out of the reach of children!

MANUFACTURERS RESPONSIBILITY

Our products have been developed for professional use in dentistry. As the application of our products is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our products in accordance with the applied standards.

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