



	Sterilizable in a steam sterilizer (autoclave) at the temperature 134°C(273°F)
	The device has not been subjected to a sterilization process
	Medical Device
	Consult instructions for use
	Washer-disinfector for thermal disinfection

P.16

	Serial number
	Conforms to CE European Directive of "Medical device directive 93/42/EEC"
	Authorized representative in the European Community.
	Batch code
	Date of manufacture
	Manufacturer

TRHVRR

Malfunction	Cause	Solution
High pitched or unusual noise, reduced cutting power or failure to rotate	Debris in handpiece or gear mechanism	Clean and lubricate handpiece

P.17

Reduction in speed	Motor problem	Check motor speed
Handpiece will not engage bur or hold bur in chuck	Damaged chuck	Send device to Precision Plus authorized repairer
Abnormal bur rotation or failure to cut	Bearing damage	Replace damaged components

If the handpiece will not work correctly after following above troubleshooting solutions, please contact Precision Plus or an authorized maintenance centre.

DD

Precision Plus products are warranted against manufacturing and material defects. Precision Plus reserves the right to analyze and determine the cause of any problem. The warranty is voided should the product not be used in accordance to the user manual, not for its intended purpose, has not been repaired by Precision Plus or an authorized Precision Plus repairer, or has non genuine Precision Plus parts installed. Replacement parts will be made available for seven years after the date of discontinuation of the model.

P.18

MSRVDRBRGE

In order to avoid the health risks to operators handling the disposal of medical equipment, as well as the risks of environment contamination caused thereof, a surgeon or a dentist is required to confirm the equipment is sterile. Ask specialist firms who are licensed to dispose of specially controlled industrial wastes, to dispose of the product for you. Always follow your country specific laws, directives, standards and guidelines for disposal of medical devices.



manufactured by:
Precision Plus China Limited
Flat 103, No. 122, Tianqiao Road,
Tutang, Changping, Dongguan,
Guangdong, &LD
Tel: +86-769-82306233
Fax: +86-769-82301566



imported and distributed by:
Premium Plus Poland sp. z o.o.
ul. Bukowska 27
62-081 Wysogotowo
oland
Tel: 48-61-880-10-94
Email: info@premiumpluspl.com

L20-1L 12/2021

made in China