

USER INFORMATION

CAREFULLY READ THESE USER INSTRUCTIONS BEFORE USING THIS FACE SHIELD

Applicable to products:	1145 1146	Face Shield Kit Replacement Shields
Manufacturer:	Premium Plus (Dongguan) Limited No. 1 Industrial Area, Tutang, Changping, Dongguan, Guangdong, People's Republic of China	
EU Notified body:	Module B and C2 - SATRA Technology Europe Ltd, Bracetown Business Park, Clonee, D15YN2P. Republic of Ireland (Notified Body 2777).	
UK Approved body:	Module B and C2 - SATRA Technology Centre Ltd, Wyndham Way, Telford Way, Kettering, Northamptonshire, NN16 8SD, U.K. (AB0321).	
Declaration of conformity:	www.premiumplushk.com	

These products are classed as Category III Personal Protective Equipment (PPE) by the European PPE Regulation (EU) 2016/425 and PPE Regulation (EU) 2016/425 as brought into UK Law and amended and have been shown to comply with this Regulation through limited testing against the Harmonised European Standard and Designated Standard EN 166:2001 (see below).

This product is designed to provide some protection against contact with airborne droplets that may constitute a biological hazard. **However, always remember that no item of PPE can provide full protection and care must always be taken while carrying out the risk-related activity. Never use this visor for protection against other hazards for which the product is not design for or tested against**

PERFORMANCE AND LIMITATIONS OF USE – These products have been tested in accordance with EN 166: 2001 Clauses: 6 | Design; 7.1.1 | Field of vision; 7.1.2.1 | Refractive powers; 7.1.2.2 | Transmittance; 7.1.2.3 | Diffusion of light; 7.1.3 | Quality of materials; 7.1.4.2 | Increased robustness; 7.1.5.1 | Stability at elevated temperature; 7.1.7 | Resistance to ignition; 7.2.4 | Droplets and splashes of liquids

INSTRUCTIONS FOR USE –

<p>For personal protection against droplets and splashes</p> <ol style="list-style-type: none"> 1. Before assembly, peel protective film from both sides of the large and small shields. 2. Attach the larger shield to the frame using the attachment lugs in the visor frame. 3. Push the smaller shield through the opening in the peak of the visor frame, circular side down. Use the grooves to secure shield to the visor frame. 4. Place on head and adjust the cinch cord for a secure fit. <p>For re-use please refer to section “CLEANING” below.</p>

FITTING AND SIZING – These products are one size fits all. Only wear products of a suitable size that are correctly adjusted. Products which are too loose may not stay in place and if too tight will be uncomfortable. To put on and take off products:

1145/ 1146	1. Place on head and adjust the cinch cord for a secure fit.
-----------------------	--

COMPATIBILITY – To optimise protection, it will be necessary to use these products with suitable gloves/gowns/masks. In this case, before carrying out the risk-related activity, consult your supplier or supervisor to ensure that all your protective products are compatible and suitable for your application. Warning - Parts of the visor that come into contact with your skin may lead to allergic reactions. In this case please discontinue use and seek medical advice.

STORAGE AND TRANSPORT – When not in use, store the visor in a well-ventilated area away from extremes of temperature. Never place heavy items on top of it. If possible, avoid excessive folding, do not place shields face down on surfaces and preferably store it hanging vertically. If the product is wet, allow it to dry fully before placing it into storage. Store in a dark place out of direct sunlight.

OBSCOLESCENCE – This visor has a shelf life of 3 years. Service life will depend on usage conditions but the visor should be disposed of and replaced if badly scratched or damaged in some way that prevents safe use.

REPAIR – If the shields or frame becomes damaged, it will NOT provide the optimum level of protection, and therefore should be immediately either replaced. Replacement shields product code 1146. Never use damaged products.

CLEANING – Only use the following recommended cleaning treatment.

1. Disassemble the shields from the frame.
2. Rinse heavy debris off components.
3. Use a 2 in 1 surface cleaner or wipe to clean and disinfect all components. NEVER use solvent based cleaning agents.
4. Ensure all components are thoroughly wiped down using the disinfectant.
5. After cleaning, wipe off any excess disinfectant and dry components using a clean paper towel or microfibre cloth.

Premium Plus (Dongguan) Ltd.				
File No.:	File Name:	Device:	Version:	Page:
TF-PPE-02k	User Information	Reusable Face Shields	V2.3	1 of 1